

CEREC Optispray

All the benefits at a glance.

■ Simple

CEREC Optispray replaces conventional two-component systems liquid and powder. Now everything you require is all-in-one without the necessity for installation or an extra unit. Simply coat the tooth, trigger the camera – and the optical impression is ready. The flexible nozzle can be bent and rotated to the chosen position.

■ Precise

The blue colour clearly indicates the area that has been sprayed. Unlike powder products, CEREC Optispray can be applied very thinly. This means that the camera can acquire very detailed images of the preparation – a precondition for restorations that fit perfectly.

■ Hygienic

Each new patient receives a new spray nozzle. Effective infection control couldn't be any easier. The nozzle has been specifically designed to prevent blockages. CEREC Optispray is always ready for use.

■ Pleasant taste

The mild mint flavour of CEREC Optispray is popular with patients. It avoids the uncomfortable feeling that they are receiving medication. Last but not least, the easy-to-use CEREC Optispray simplifies and speeds up the treatment process.



CAD/CAM SYSTEMS | INSTRUMENTS | HYGIENE SYSTEMS | TREATMENT CENTERS | IMAGING SYSTEMS

SIRONA – UNIQUE WORLDWIDE SYSTEMS EXPERTISE IN DENTAL EQUIPMENT PRODUCTS

Sirona develops and manufactures a comprehensive range of dental equipment, including CAD/CAM Systems for dental practices (CEREC) and laboratories (inLab), Instruments and Hygiene Systems, Treatment Centers and Imaging Systems. Sirona manufactures high technology products that guarantee ease of use and a high return on investment – for the good of your practice and for the benefit of your patients. In this way, you can approach every challenge that you face, confident in the knowledge that: **It will be a great day. With Sirona.**

Sirona Dental Systems · E-mail: contact@sirona.com · www.sirona.com

sirona.
The Dental Company



CAD/CAM SYSTEMS | INSTRUMENTS | HYGIENE SYSTEMS | TREATMENT CENTERS | IMAGING SYSTEMS

CEREC Optispray – SIMPLE, EXACT, HYGIENIC Precision at the touch of a button.

Subject to technical changes and errors in the text. Order No. A91100-M43-B1 59-01-7600, Printed in Germany, Dispo No. 04605, 4023/14091 WS 11086.V0

sirona.
The Dental Company

CEREC Optispray – REMARKABLY UNCOMPLICATED

Simple handling, perfect results.

Easy to use, patient-friendly, result oriented and ultra-precise: CEREC Optispray embodies all the key features of the CEREC system. Forget about labour-intensive scan powders and liquids. Creating optical impressions with CEREC Optispray is now very quick and convenient. In addition, CEREC Optispray delivers convincing benefits in terms of precision and hygiene. **It will be a great day. With Sirona.**




CEREC Optispray

The easy way to create precise optical impressions.

1. Attach nozzle

The hygienic disposable nozzle is flexible and resistant to blockages. Easy to attach, the nozzle can be bent and rotated in all directions, facilitating easy access to the upper and lower jaws.

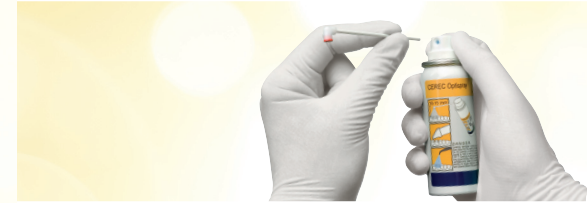

2. Shake

Shaking the can is a prerequisite to achieve the thinnest possible coating. The quick-drying spray adheres instantly to the tooth. The dentist only has to ensure the preparation is dry before applying CEREC Optispray.



3. Spray

The distinctive blue colour of CEREC Optispray is easily visible. As it can be applied very thinly, excessive accumulations of powder can be avoided.



4. Acquire image

CEREC Optispray applies a homogeneous anti-reflective coating on the tooth surface. The camera is then able to acquire a high-precision optical impression without any distortion. This enhances the accuracy of the 3D model.



5. Rinse

Compared to other coating materials, CEREC Optispray applies a much thinner layer and is water-soluble. After the optical impression has been acquired, CEREC Optispray is quick and easy to remove – for example, with the SPRAYVIT syringe.

